Discovery Center Remote v 2.0



Freeze button: Works the same way the "Capture" button does on the camera.

Recall: Lets you view the images you've just captured with the camera.

Up/Down: Enables you to scroll through the images via recall button.

Single square/Four square (quad mode)

Delete: Enables you to delete the images you do not want

Neg: Brings up the Negatoscope Mode.

B/W: Turns the screen black & white for x-ray film capturing

VR. Vibration reduction. If the image you captured is kind of "shakey", you can press this button while in recall mode to eliminate it.

SD Card Functions:

Sav: While you are in recall mode and you see an image that you'd like to print or save, hit this button and it will save the image onto the SD Memory card

< and >: These two buttons let you scroll through the images that are ON THE MEMORY CARD, you will see "Playback Mode" at the top of the screen

OSD: On Screen Display. Lets you enable/disable the caption at the top while you ar are in Capturing Mode or Playback mode.

Setup: You can set the time, date, and reformat the SD card in set up

Delete: lets you delete the images you do not want int he SD Memory Card

For more Technical Support information, visit http://www.RFAmerica.net