


EXTRAORAL CAMERA USER'S MANUAL

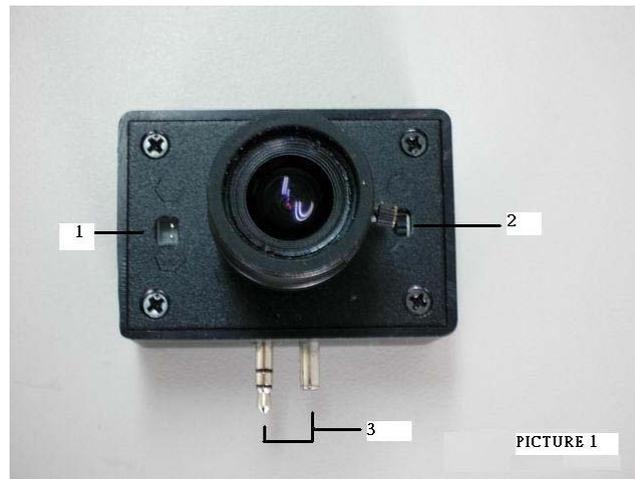

Description for Picture 1 :

1. Light diffuser connection port.
2. Light diffuser power switch-----switch to upper side for power OFF and lower side for power ON.
3. Connection plug to bottom of Discovery wireless Intraoral Camera handpiece.




Description of Picture 2 :

4. Button to Freeze the images.
5. Pole to adjust lens focal for near range or far range (this pole is fixed).
6. Pole for focusing object's image (slightly adjust to get the focus).



Connect diffuser light according to Picture 3.
Be sure the direction of connector position when plugging.



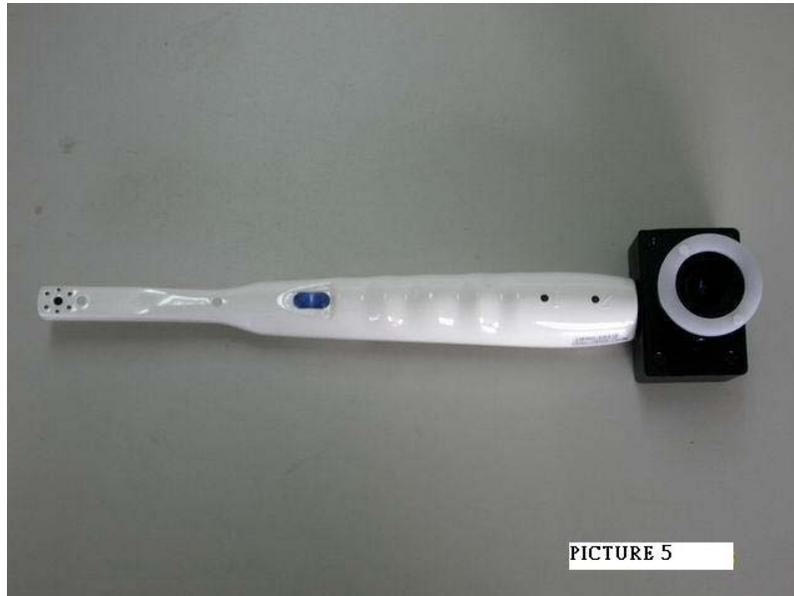

Plug the Extraoral Camera to the bottom of Discovery wireless intraoral camera handpiece. Be sure to plug Extraoral camera in right direction. (refer to Picture 4 and Picture 5).



Move the switch on Discovery wireless Intraoral Camera handpiece to “EXTRA” position and the Extraoral camera will ready to use.

Note : Pole no 6 (Picture 2) is fixed, so user slightly adjust pole no 5 only to focusing images. Object range from 5mm to ∞ .

After get better image,press the FREEZE button to freeze image.

Move the switch to “INTRA” position when user want to use Intraoral Camera (Picture 7).